

Midas Rex® ESP Plus

*Proactive + Preventive = Performance
Improving Procedure Efficiency*



Increased Operating Room Efficiency

Equipment availability and performance are key drivers to surgical procedure efficiency. Medtronic's ESP Plus program provides overnight exchanges and scheduled onsite planned maintenance (PM) visits. Medtronic factory-trained technicians will come to your facility at a pre-arranged date and time. They will inspect and test your Midas Rex motors, attachments, consoles, and foot controls.

- Increased equipment uptime and less case disruption with overnight exchanges.
- Performance and compliance assurance with regularly scheduled onsite PM visits by Medtronic factory-trained technicians.
- Recommendations for your central sterile staff on proper care, cleaning and handling of your Midas Rex system.
- All shipment costs included.
- Priority seating for Medtronic Midas Rex Institute training events.

What's Included in a PM Visit

Planned maintenance at your location on each motor, power console, foot-control, attachment, hose and cable by Medtronic factory-trained technicians.

- Visual, audible and tactile inspection for excessive wear and tear.
- Functional tests on motors, power consoles, foot-controls and attachments.
- Testing with thermal and optical meters to validate proper operating specifications.
- Installation of latest software onto power consoles.
- Immediate exchanges on equipment that does not pass inspection.
- A full PM inspection report for compliance record keeping.
- Training for central sterile staff to ensure proper cleaning and sterilization.



Planned Maintenance Schedule

Maintenance Level	# of Tools Used per Motor per Year	Usage Rate per Motor per Year	PM Level
High	150-200	> 12	Every 3 months
Med High	100-149	8-11	Every 4 months
Med Low	50-99	4-7	Every 5 months
Low	< 50	< 4	Every 6 months

For more information contact your local Medtronic Neurosurgery area sales manager or go to www.responsecare.medtronic.com.

**Medtronic plc
Neurosurgery**
4620 North Beach Street
Fort Worth, Texas 76137
USA
Toll free: (800) 433-7080
Telephone (817) 788-6400
www.medtronicneurosurgery.com



Caution: Federal law (USA) restricts this device for sale by or on the order of a physician. Refer to product instruction manual/package insert for instructions, warnings, precautions, and contraindications. Midas Rex® and ResponseCare® are registered trademarks of Medtronic, Inc.